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1. OBJETIVO

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ANATOMIC WEDGES

1. PURPOSE

To establish the product features, its *application, presentation, legal and normative information, in order to serve as a standard for elaborating labels, instructions for use, advertising prints and electronic communication.*

2. PRODUCT IDENTIFICATION

2.1 Commercial name

Anatomic Wedges

2.2 Technical name

Interdental wooden wedges

2.2.1 Technical name ANVISA

Dental Wedge

2.2.2 Code of the technical name, in compliance with ANVISA

2101015

2.2.3 Product Classification in compliance with ANVISA

Rule 5, Class I - Intrinsic medical products applicable to orifices that are intended for transient use and which are not intended to be connected to an active medical device.

2.3 General description

TDV wooden wedges are indicated for general use in restorative procedures, with the mechanical function of adapting matrices and/or separating adjacent teeth. The wooden wedges are available in five different sizes and do not require cuts or adjustments, because its anatomical format allows an easy insertion into the interproximal areas and ensures the fixation of the matrices.

2.4 Brief description

Wooden wedges, easy to adapt in the interproximal space. Available in five sizes codified by colors.

2.5 Features and benefits

- *Anatomically shaped for a better adjustment to the interproximal area*
- *Dyed with non-toxic pigments, does not stain the restoration.*
- *Available in 5 sizes*

2.6 Presentation

According EPA 0004020000.pt

3. TECHNICAL INFORMATION

3.1 Composition

Natural: Wood
Colored: Wood and *non-toxic* reactive dyes.

3.1.1 Qualitative and Quantitative Formula

Natural: Wood: 100%
Colored: Wood: 99%
Non-toxic reactive dyes: 1%

3.1.2 Function of each component

Natural: Wood: Structural
Colored: Wood: Structural
Non-toxic reactive dyes: Color

3.2 Physical principle of operation

The anatomical format of the wedge allows its easy insertion into the interproximal areas. The wedge provides a better adjustment of the matrix and separation of the adjacent teeth.

3.3 Indication

TDV wooden wedges are indicated to general use, in restorative procedures using matrices.

3.4 Sizes

Available in 5 sizes.

3.5 Storage

To be stored in dry and fresh place, protected from sunlight.

3.6 Use

- Select the most appropriate wedge size for the procedure.
- Insert the wedge into the interproximal space.

3.7 Warnings, precautions and restrictions

Professional use only.

Single use only. Its reuse may cause cross contamination

If there is proven allergy to any of its components, the product must not be used.

To avoid an accidental swallowing of this product, it is necessary the application of a total isolation technique of the operatory field. The product has a sharp point and must be handled in order not to cause lesions to the patient.

3.8 Expiration date

Considering the deterioration of the packaging validity period is of 5 years.

3.9 Symbols

4. PRODUCT DRAWINGS

DP 02.04.02.00, DP 02.04.02.01, DP 02.04.02.02, DP 02.04.02.03, DP 02.04.00.15 and DP 3.8109.

5. MANUFACTURING FLOW

Check manufacturing details FF 00.04.02.00.

6. DESCRIPTION OF EFFECTIVENESS AND SAFETY

Check risk management.

7. REFERENCES

Not available

8. VISUAL IDENTIFICATION STANDARD

PRINTED	COLOR	PAPER
Instructions for use	C = 0 M = 0 Y = 0 K = 80	Sulphite Paper 56g/m ² Two-sided printing
Blister Paper Card	C = 100 M = 73 Y = 10 K = 48 C = 0 M = 87 Y = 84 K = 0 C = 0 M = 0 Y = 0 K = 80	Couché 90g/m ²

PRIMARY PACKAGING

PET Packaging

9. DISTRIBUTION OF INFORMATION BY DOCUMENT TYPE

	Instructions for use	Blister paper card	Catalogue	Mini catalogue	Website	Brochure
TDV Dental Ltda. – CNPJ 81.591.786/0001-60 XV de Novembro, 9944 - CEP 89107-000 Pomerode/SC - Brasil ☎ 55 47 3395-6115 info@tdv.com.br – www.tdv.com.br  Obelis S.A. Bd. Général Wahis 53 1030 Brussels, Belgium Tel: + (32) 2 732 59 54 E-mail: mail@obelis.net	X	X	X	X	X	X
Commercial name (2.1)	X	X	X	X	X	X
Technical name (2.2)	X	X	X	X	X	X

	<i>Instructions for use</i>	<i>Blister paper card</i>	<i>Catalogue</i>	<i>Mini catalogue</i>	<i>Website</i>	<i>Brochure</i>
Technical name ANVISA (2.2.1)	X	X				
General description ¹ (2.3)	X	X	X		X	
Brief description (2.4)				X		
Features and benefits (2.5)			X			X
Reference (2.6.1), code (2.6.3) and content (2.6.4)	X	X	X		X	
Barcode (2.6.5)	X	X				
Composition (3.1)	X	X			X	
Physical principle of operation 2 (3.2)	X	X			X	
Batch number and validity (see production order)	X	X				
Indication ^{1 2} (3.3)	X	X	X		X	
Sizes ² (3.4)	X	X	X		X	
Storage (3.5)	X	X			X	
Use (3.6)	X	X			X	
Warnings, precautions and restrictions (3.7)	X	X			X	
Expiration date (3.8)	X	X				
Symbols (3.9)	X	X				
Tec. Resp.: Taís Cúgola. Coelho Pereira – CRF/SC 13884	X	X				
ANVISA Registration n°10291220032	X	X			X	
References (7)					X	
Date of publication or last revision of the instructions for use	X	X				

10. APPROVAL

Silvio Cesar da Costa
Management Representative

José Floriano Souto
Commercial Manager - Domestic Market

Taís Cúgola Coelho Pereira
Technical responsible – CRF/SC 13884

¹ For catalog and website, it is a marketing decision the use of “indication” or “general description”

² Information already available in the product’s general description, thus it is not necessary to include this item.

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